


ROGER MEDIA HUB CHEAT SHEET

Connecting

Turn On

Press and hold on/off button for at least 3 seconds until you get green flashing light on device.

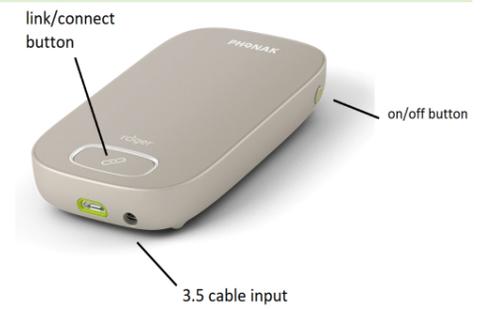


Connect to Audio Source

Connect the MediaHub to the Wallplate using the 3.5 audio cable. You can connect direct to computer using the headphone jack of computer/tablet or phone.



Connect to Receiver / Mylink


Press the link/connect button and hold within 5 cm to either the MyLink or Roger X receiver. Student will hear beep upon connection.

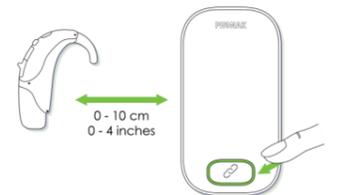

Compatible Devices

1. Roger device must be turned on to connect
2. Hold Roger Touchscreen Mic within 10cm
3. Tap **connect** to add device network

Roger MyLink

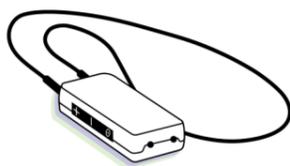

Roger X Receiver



Roger Touchscreen



Troubleshooting

Indicator Lights



- On/Off
- Charging
- Fully Charged
- Switched On
- Recharge Device

- On/Off
- Charging
- Fully Charged
- Audio Broadcast
- Connecting
- Connection Successful
- Connection Failed
- Primary Device Not available (When paired with a touchscreen)

Tips

- Ensure that cables are connected equipment is connected with 3.5 cables from: Projector or TV
- Check cable is connected at wall plate front and back.
- Volume is up on all equipment that is being used.